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System Configuration

General Archive Settings Folders Xpress Settings Host Communication Settings

2 System Name serial number- lab name-city-country

Date Format DD/MM/YY

Time Format 24 Hour (HH:mm:ss)

Use Patient ID

Scan Patient ID Barcode

Scan Sample ID Barcode

Scan Cartridge Barcode

Require Start Test Login

Never

Always

Start Test Login Timeout (In Minutes) 28

3

Audio At End Of Test

Print Test Report At End of Test

Assay Statistics Deletion Reminder

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The screenshot shows the GeneXpert Dx system software interface. The 'Setup' menu is open, and the 'User Administration' option is selected. A red arrow points to the 'Setup' menu. Below the main interface, a 'User Administration' dialog box is open, showing a table with columns for 'UserName', 'Full Name', and 'Type'. The 'Add' button at the bottom of the dialog box is circled in red, and a red arrow points to it.

Module Name	Assay	Sample ID	Progress	Status	Remaining Test Time
-------------	-------	-----------	----------	--------	---------------------

Sample ID	Mod Name	User	Result	Assay	Status	Error Status	Start Date
-----------	----------	------	--------	-------	--------	--------------	------------

UserName	Full Name	Type
----------	-----------	------

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4. Seleccione **User type**: “Admin”

5. Clic en “OK”

The screenshot shows a dialog box titled "Add User" with the following fields and options:

- User Name: [Empty text box]
- Full Name: [Empty text box]
- Password: [Empty text box]
- Confirm Password: [Empty text box]
- User Type: A dropdown menu with the following options: Basic (selected), Basic, Detail, Research, and Admin (circled in red).
- Buttons: OK (circled in red) and Cancel.

Red arrows point from the text instructions to the corresponding elements in the dialog box: one to the User Name field, one to the Admin option in the User Type dropdown, and one to the OK button.

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1. Clic en **“Setup/User administration”**

The screenshot shows the GeneXpert Dx System software interface. The 'Setup' menu is open, and the 'User Administration' option is highlighted. A red arrow points to this option. Below the main interface, the 'User Administration' dialog box is open, showing a table with columns for 'User Name', 'Full Name', and 'Type'. A red arrow points to the 'Add' button at the bottom of the dialog box.

User Name	Full Name	Type
-----------	-----------	------

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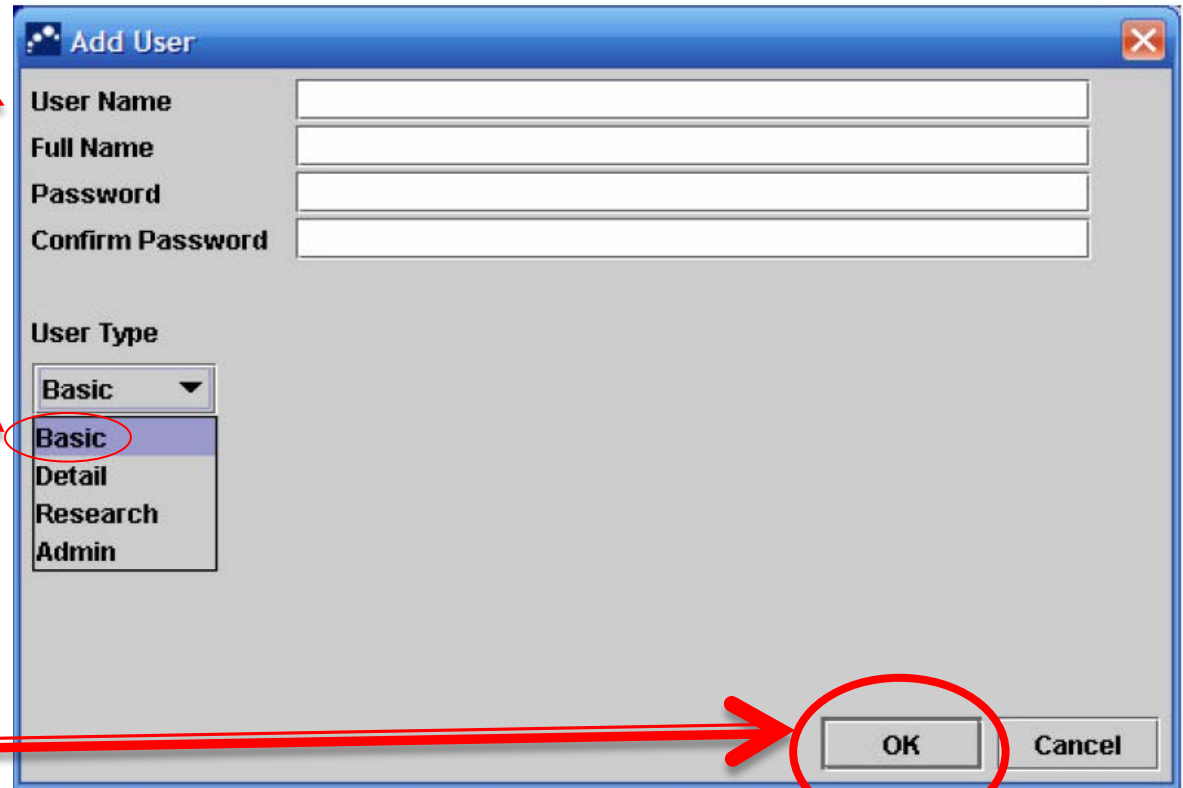
5. Clic en "OK"

The screenshot shows the 'Add User' dialog box with the following fields and options:

- User Name: [Text Input]
- Full Name: [Text Input]
- Password: [Text Input]
- Confirm Password: [Text Input]
- User Type: [Dropdown Menu]

The 'User Type' dropdown menu is open, showing the following options: Basic, Detail, Research, and Admin. The 'Basic' option is highlighted and circled in red. A red arrow points from the text '4. Seleccione User type: "Basic"' to this option. Another red arrow points from the text '5. Clic en "OK"' to the 'OK' button, which is also circled in red.

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